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USER INFORMATION. PLEASE READ THIS INFORMATION CAREFULLY.

# LACALUT® FIX

FOR DENTURES

NEUTRAL | MINT

## ADHESIVE CREAM EXTRA STRONG EN | MEDICAL DEVICE

Read the entire package insert carefully, as it contains important information for you. This medical device is available without prescription. However, in order to achieve the best possible treatment success, the Lacalut Fix adhesive cream must be used according to the instructions.

**Dosage form and content:** Adhesive cream, 40 g

**Use:** Adhesive cream for application on dental prostheses.

**Precautions and special instructions:** Use only on healthy (undamaged) oral mucous membranes.

**Dosage instructions, type and duration of application:** Thoroughly clean and dry the denture and apply Lacalut FIX adhesive cream extra strong at evenly spaced intervals. Then apply the adhesive cream in the hollow of the upper/lower denture and insert. Press the denture firmly against the jaw for a few seconds. Wait about 5 minutes before eating or drinking. The dosage can be adjusted individually. Close the tube carefully after use. The cream can be applied to both dry and wet dentures. Do not put adhesive cream directly into the mouth.

**Contraindications:** Do not use where there is hypersensitivity to any of the ingredients.

**Interaction with other products:** None yet known.

**Side effects:** In isolated cases, hypersensitivity reactions of the mucous membranes are possible. If you observe side effects, inform your doctor or pharmacist.

**Keep out of the reach of children!**

**The adhesive cream should not be used after the expiry date  printed on the folding carton and tube.**

**Reporting of side effects:** If you experience any side effects not listed in this leaflet, they can be reported directly to the Federal Institute for Drugs and Medical Products at: Bundesinstitut für Arzneimittel und Medizinprodukte, Abt. Pharmakovigilanz, Kurt-Georg-Kiesinger-Allee 3, D-53175 Bonn, Web: <http://www.bfarm.de> or the manufacturer.

**Did you like the product?** For suggestions or requests please contact the manufacturer.

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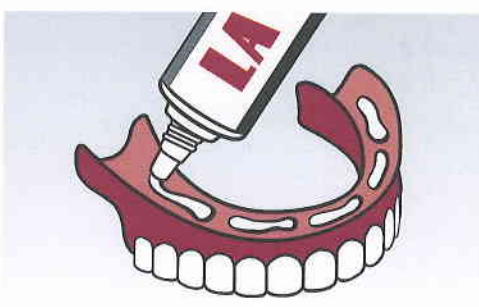
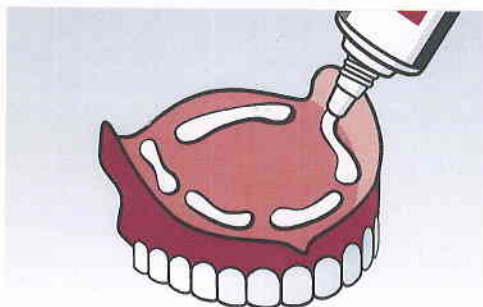
**Applications:** Either apply the LACALUT® FIX adhesive cream in dots or thin strips, not too close to the edge of the denture

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Dr. Theiss Naturwaren GmbH